### **STERISHEET 355 SMS**


## STERISHEET

#### DESCRIPTION

Sterisheet sterilization wraps are best in class Sterile Barrier Systems for CSSDs in hospitals and clinics. Sterisheet SMS products are available as interleaved, bonded, or single sheets.

#### **COMPOSITION**

100% synthetic fibers made of polypropylene.
Spunbond / Meltblown / Spunbond
Spunbond network consists of long, strong, and thick polypropylene filaments.
Meltblown layers are made of short and thin polypropylene microfibers.

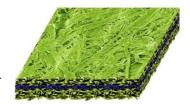

#### SUITABLE FOR STERILIZATION METHODS

- o Steam
- **EO**
- Vaporized hydrogen peroxide (H2O2)

#### **APPLICATION**

For medium-size and heavy trays and instrument kits.

#### SIZES AVAILABLE

Choose standard sizes to optimize your costs. Other sizes are also available upon request.

W

W

75x75 cm 90x90 cm

100x100 cm 120x120 cm



sterime<u>d</u>.

STERISHEET ® IS A BRAND OF



**STERISHEET 355 SMS** 



### 2|4)

#### **COLORS**



#### PERFORMANCES

Mechanical and bacterial barrier properties are internally tested on a routine basis according to regulatory requirements of all of our products. Combining them with random tests conducted by external and accredited laboratories leads Sterimed to secure the best performances on Sterisheet products.

#### **Mechanical Properties**

Preservation of pack integrity from closure till the point of use depends on the material's resistance to tearing, puncturing, and breaching stresses generated all along the distribution with the hospital. Any mechanical weaknesses will increase the risks of event-related ingression of microorganisms into the pack.

Excellent mechanical properties will provide you additional safety while using our materials. Optimal strength and resistance provided in every sheet.

PROPERTIES	TYPICAL
Basis Weight	55 g/m²
Air permeability	90 l/min/dm2
Thickness	385 µm
Hydrost Test	45 mBar
Tensile strength MD	2.4 kN/m
Tensile strength CD	1.1 kN/m
Elongation MD	55 %
Elongation CD	55 %
Burst	260 kPa
Tears MD	8000 mN
Tears CD	10000 mN

#### **Bacterial Properties**

Sterilization wraps must prevent microorganisms' ingression inside the package.

To reach this performance, different types of testing have to be performed on the products to reproduce both kinds of ingression vehicles possible:

• Airborne ingression

• Waterborne ingression

For instance, according to the TNO final pack test method, validating the materials & the folding technique, STERISHEET 355 SMS exhibits a BARRIER PROTECTION > 99,99%



**STERISHEET 355 SMS** 



#### **COMPLIANCE TO STANDARDS**

Sterisheet products range is classified as Class I Medical Device according to the European Medical Device Regulation (MDR 2017/745/EU). Its CE marking illustrates the relevant compliance.

Sterisheet products conform with the standards below:

EN ISO 11607-1

EN 868-2

#### **OUR WRAPS MANUFACTURING THIRD PART CERTIFICATION**

ISO 13485 standards

#### **PACKAGING PRIOR TO USE**

Sheets display is optimized by adjusted folding depending on the size and type of the product. We have carefully tested the best solution to ensure the most convenient handling for end users.

• PRIMARY TRANSPORT PACKAGING

Number of sheets is maximized and wrapped in a transparent polyethylene bag with a quick product ID.

• SECONDARY TRANSPORT PACKAGING

Secondary packaging is a neutral brown color cardboard box providing transportation stress resistance.

#### LABELING

Product traceability is fully insured through labeling according to regulations on each transport packaging.

#### **STORAGE CONDITIONS**

Sterimed recommends the following storage conditions for best performance of sterilization wraps: Storage in a cool, dry location away from direct exposure to natural light, strong artificial light & UV sources.



### **STERISHEET 355 SMS**

### STERISHEET

Cardboard boxes should never be stored in direct contact with the floor. Storage of the products shall be done in areas that are not subject to extreme temperature changes such as in contact with heated objects, vents, or cold walls. As per AAMI ST79 "*Comprehensive guide to steam sterilization and sterility assurance in health care facilities*" recommendations: Before use, hold packaging materials at room temperature (20°C to 23°C) and at a relative humidity ranging from 30% to 60% for a minimum of 2 hours is a good practice for optimum use performances.

#### **USE BY DATE**

Provided the above storage conditions are met, the upper limit of the time interval during which the performance characteristics of the sterilization wrap are demonstrated is 5 years of the manufacturing date.

#### **ENVIRONMENTAL IMPACT & WASTE MANAGEMENT**

Oil derived product, renewable content 0%.

Disposal as per local regulations after use.

Product Data Sheet - STERISHEET 355 SMS | VERSION 3 | Edited on July 14 - 2021

STERISHEET ® IS A BRAND OF



www.sterimed.fr